



## INTENDED USE


The multi-panel drug screening device in a cassette is a one-step in vitro test device consisting of drugs of abuse screen test strips. It is designed for qualitative determination of drug substances in human urine specimens. This assay may be used in the point of care setting.




Remove cap



Dip into urine sample for 10 seconds

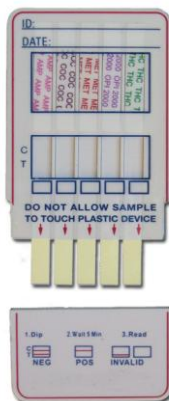

Replace cap

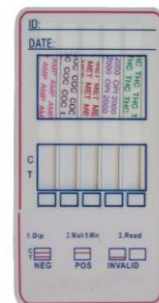

Wait 5 minutes to interpret results

## Available from 2 up to 12 drug analyses combinations

### Drug Analyses Choices:

Parameter	Callibrator	Cut-off (ng/mL)											
		10	25	50	100	150	200	300	500	600	1000	2000	
AMP (Amphetamine)	Amphetamine							✓	✓			✓	
BAR (Barbiturates)	Secobarbital							✓					
BZO (Benzodiazepines)	Oxazepam				✓		✓	✓					
BUP (Buprenorphine)	Buprenorphine-β-3-D-glucuronide	✓											
COC (Cocaine)	Benzoylgonine				✓		✓	✓					
COT (Nicotine)	Cotinine						✓			✓			
EDDP (Methadone metabolite)	2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine				✓								
FYL (Fentanyl)	Fentanyl	✓											
KET (Ketamine)	Ketamine											✓	
MDA (Methylenedioxyamphetamine)	Methylenedioxyamphetamine								✓				
MDMA (Ecstasy)	3,4-Methylenedioxymethamphetamine							✓	✓			✓	
MTD (Methadone)	Methadone							✓					
MET (Methamphetamine)	Methamphetamine							✓	✓			✓	
MOR (OPI) (Morphine/Heroin)	Morphine				✓		✓	✓					✓
MQL (Mandrax)	Methaqualone							✓					
OXY (Oxycodone)	Oxycodone				✓								
PCP (Phencyclidine)	Phencyclidine		✓										
PPX (Propoxyphene)	Norpropoxyphene							✓					
TCA (Tricyclic antidepressants)	Nortriptyline											✓	
THC (Marijuana)	11-nor-Δ <sup>9</sup> -THC-9-COOH		✓	✓		✓	✓		✓				
TML (Tramadol)	Tramadol				✓								

## Drug Testing Africa

+27 21 785 5848

+27 86 518 2210

[info@drugtesting.co.za](mailto:info@drugtesting.co.za)

[www.drugtesting.co.za](http://www.drugtesting.co.za)

