

AccuTest™

International Gold Standard for drug screening

Single Panel Drug Screening Device in Cassette

FOR THE QUALITATIVE ASSESSMENT OF A DRUG AND ITS METABOLITE IN HUMAN URINE

- Easy to use
- Accurate and reliable
- Instant results



- FDA APPROVED
- CLIA WAIVED
- CE MARKED

All batches independently tested to assure the products meet our on-going high quality standards



The preferred supplier of high quality sophisticated and innovative diagnostic assays

INTENDED USE

The multi-panel drug screening device in a cassette is a one-step in vitro test device consisting of drugs of abuse screen test strips. It is designed for qualitative determination of drug substances in human urine specimens. This assay may be used in the point of care setting.




Put cassette flat on a flat surface



Suck up urine with dropper



Dispense 3 drops into sample well

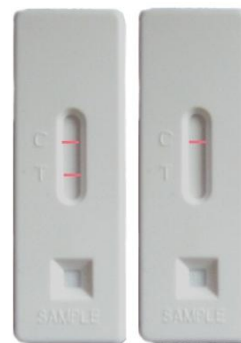

Wait 3 minutes but no more than 10 minutes to read results

Available in any of the following substances

Drug Analyses Choices:

Parameter	Callibrator	Cut-off (ng/mL)											
		10	25	50	100	150	200	300	500	600	1000	2000	
AMP (Amphetamine)	Amphetamine							✓	✓			✓	
BAR (Barbiturates)	Secobarbital							✓					
BZO (Benzodiazepines)	Oxazepam				✓		✓	✓					
BUP (Buprenorphine)	Buprenorphine-β-3-D-glucuronide	✓											
COC (Cocaine)	Benzoylgonine				✓		✓	✓					
COT (Nicotine)	Cotinine						✓			✓			
EDDP (Methadone metabolite)	2-Ethylidine-1,5-dimethyl-3,3-diphenylpyrrolidine				✓								
FYL (Fentanyl)	Fentanyl	✓											
KET (Ketamine)	Ketamine											✓	
MDA (Methylenedioxyamphetamine)	Methylenedioxyamphetamine								✓				
MDMA (Ecstasy)	3,4-Methylenedioxymethamphetamine							✓	✓			✓	
MTD (Methadone)	Methadone							✓					
MET (Methamphetamine)	Methamphetamine							✓	✓			✓	
MOR (OPI) (Morphine/Heroin)	Morphine				✓		✓	✓					✓
MQL (Mandrax)	Methaqualone							✓					
OXY (Oxycodone)	Oxycodone				✓								
PCP (Phencyclidine)	Phencyclidine		✓										
PPX (Propoxyphene)	Norpropoxyphene							✓					
TCA (Tricyclic antidepressants)	Nortriptyline											✓	
THC (Marijuana)	11-nor-Δ ⁹ -THC-9-COOH		✓	✓		✓	✓		✓				
TML (Tramadol)	Tramadol				✓								

Drug Testing Africa

+27 21 785 5848

+27 86 518 2210

info@drugtesting.co.za

www.drugtesting.co.za

